### DE Anwendung

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### EN How to use

1. Unfold the mask only by bending the nose clip, not by pulling the ear loops. Please make sure that the nose clip is fully bent open (at a 180° angle). Check for damage and contamination before use.
2. Hold the mask by the ear loops with both hands. Place your chin in the mask first, then pull the mask over your nose. The nose clip should be positioned just above the nose bridge. Pull ear loops behind your ears.
3. Adjust the nose clip with both index fingers from the center outwards.
4. Check correct fit of mask. The Mask should fit tightly and without gaps onto the face. Hold both your hands in front of the mask and exhale forcefully. If air comes out of the nose area, adjust the nose clip to suit, or press it down more.

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Particle-filtering half mask

EN 149:2001 + A1:2009 FFP2 NR

Filtration of at least 94 % of pollutants/particles from the air, up to a size of 0.6 µm

Limits of use: protection against harmful or low-toxicity and solid, aqueous and non-volatile liquid particles and aerosols, at up to 10 times the individual threshold value in accordance with DIN EN 149:2009-08

CE certification 2016/425/EU, LGA1 TECHNOLOGICAL CENTER, S. A./Applus, 08193 BELLATERRA (Barcelona) (Code Nr. 0370)

Made in Germany: Produced at TechniSat Vogtland GmbH in Schöneck, Saxony, and at TechniSat Teledigital GmbH in Staßfurt

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FFP2 respirator face mask

Part. no.: 0000/2860, 0000/2861, 0000/2862, 0000/2863, 0000/2864, 0000/2865

Storage life (in original packaging): 5 years in the sealed bag.

The full technical data sheet is available in PDF format in the download area of the TechniSat homepage at [www.technimask.de](http://www.technimask.de). The complete text of the EU Declaration of Conformity is available at the following web address: <http://konf.tsat.de/?ID=23344>

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## EN Warning and safety instructions

Follow all the safety instructions and restrictions when using this product. Non-observance may reduce its effectiveness.

Only use protective masks that are permitted for your work area. If necessary, contact the appropriate specialist for hygiene or occupational safety.

If dizziness, lack of concentration, breathing difficulties or other complaints occur during use, leave the work area immediately.

After use, dispose of the protective mask in accordance with local disposal regulations.

The protective mask is intended for single use only. Do not reuse/continue to use a protective mask that has

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already been used or is heavily soiled. Dispose of it immediately. Observe the replacement intervals for the protective mask in accordance with the occupational safety guidelines for your workplace/work area. The mask must be replaced immediately if it is damaged or if breathing resistance increases. Do not wear the mask for longer than one shift.

Facial hair can affect adequate fit.

The mask must not be used: in areas with a low oxygen content (less than 19.5%) or where there is an explosion risk, for work involving asbestos, paint spraying, or sandblasting, if you taste or smell toxins, for protection from gases or vapours, if the concentrations of toxins are unknown or are an immediate danger to life or are hazardous to health.

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